

- Take out a single test strip.
- Expose the test pad to urine for an instant, shake off any excess.
- After 15 seconds compare the colour on the pad to the colour chart on the bottle.
- The colour will tell you if there are ketones present in your urine and an approximate level.

INGREDIENTS

Ketone

Sodium nitroprusside 7.1%W/W



0 016027 495668 >


Trying to get into Ketosis?
Try our natural sweetener erythritol, it has all the taste of sugar but contains 0 calories! It's great for drinks and baking.

For more information regarding our products please head over to: www.nkdliving.com

- ✓ Results in 15 seconds
- ✓ Accurate detection
- ✓ 2 packs of 60 strips
- ✓ High quality
- ✓ Professional grade



IVD

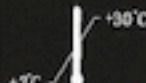

Manufactured for:
NKD Living LTD,
Unit 7a, Pineham farm, Haversham,
Milton keynes, MK19 7DP

www.nkdliving.com