Eye Wash

Multi Action



# **Multi Action Eye Wash**

Vizulize Multi Action Eye Wash has been specially developed to provide gentle and sensitive eyes.

### Warnings

If the eye bath is going to be used by more than one person the eye bath Close the bottle after use.

Do not return the used solution back to the bottle.

Do not use if allergic to any of the

If eye irritation occurs, discontinue use and consult your pharmacist or optician.

Do not swallow.

Do not use after the expiry date.



**70mm** 

# **Multi Action**

### **Directions for Use**

Eye wash can be used without having

For use in eyes only.

- 1. Wash your hands.
- 2. Rinse the eye bath with eye wash.
- 3. Fill the eye bath 1/3 full of eye wash.
- 4. Tilt your head slightly forwards and hold the eye bath at the base.
- 5. Carefully position the eye bath over your eye.
- 6. Open your eye whilst slowly raising your head.
- 7. Very gently rock your head from side

Includes an easy-to-use eye bath

pH balanced to be kind to eyes

Suitable for use with contact lenses

www.vizulize.com

### **Ingredients**

### **Storage**

Avoid direct sunlight.



## 300mle

# Cleanses & **Refreshes Eyes** Soothes Tired, Irritated,

Sensitive & Dry Eyes

**Moisture Level** 



pH balanced



**Includes** an eye bath

EC REP Advena Ltd. Tower Business Centre, 2nd Flr., Tower Street, Swatar, BKR 4013 Malta



LOT

Text revised 09/20-V1

V2 - Last amended: 07/09/2020

Pantone 354 C

Black - K

Pantone 376 C

Pantone 300 C (Lozenge)

877 Silver