Contact Lens Solution Aulti-Purpose becnevbA



Vizulize

Advanced **Multi-Purpose Contact Lens Solution**

Cleans - Rinses - Disinfects - Lubricates - Stores Vizulize Advanced Multi-Purpose Contact Lens Solution has been specially developed to bring you exceptional daily hygiene and comfort in your

than 30 days, repeat the disinfecting cycle. use and consult a Healthcare Do not use if allergic to any of the

Warnings



Advanced Multi-Purpose Contact Long

Directions for Use

To open the bottle: Twist the cap To clean lenses: Wash and dry your hands before handling contact lenses. Advanced Multi-Purpose Contact Lens Solution to each side of the lens surface and gently rub both sides for 20 seconds. Rinse the lens thoroughly Contact Lens Solution. Place the lens in the appropriate compartment of the

Ingredients

ingredients.

Storage

Store between 5°C to 30°C.

* Compared to Vizulize Multi-Purpose Soft

Superior Dual Moisturising Formula for Exceptional **Comfort*** Helps maintain the optimal performance of lenses OP ALL TL

For use with hard, soft and silicone hydrogel



Includes a lens case

EC REP Advena Ltd. Tower Business Centre, 2nd Flr., Tower Street, Swatar, BKR 4013 Malta

STERILE A

1282 Old Dalby Busine s. I F14 3NJ. UK






lenses. www.vizulize.com

Vizulize Advanced Multi-Purpose Contact Lens


Solution is suitable for use with hard and soft contact lenses as well as silicone hydrogel

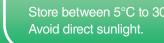


Contact Lens Solution



2

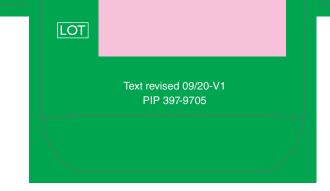

Cirrus An MPS Company	Customer	East Midlands Pharma		Colours Used
	Item Code	PEMP312		PANTONE 354 C
	Description	Vizulize Advanced Multi Purpose Contact Lens Solution 360	n a a J	REEARINTONE 3425 C
	Keyline Reference	EMP8		Black
	Varnish Reference	Drip off Primer Varnish		PANTONE 376 C
	Dimensions	70 x 70 x 180		PANTONE 311 C
Signature	Language	English		PANTONE 877 C
Warning! We cannot accept responsibility for any errors in this proof after approval. Whilst we take extreme care at all times to ensure accuracy to our client's brief, the final responsibility must be taken by our client. IF YOU SIGN THIS PROOF YOU ARE SIGNIFYING FULL APPROVAL OF DESIGN AND TEXT.	Market	UK		Keyline (Non-Printing)
	Proof By	CJE		Varnish Free (Non-Printing)
	Proof No.	2		Cirrus_Info_Box
	Date	12/11/2020		

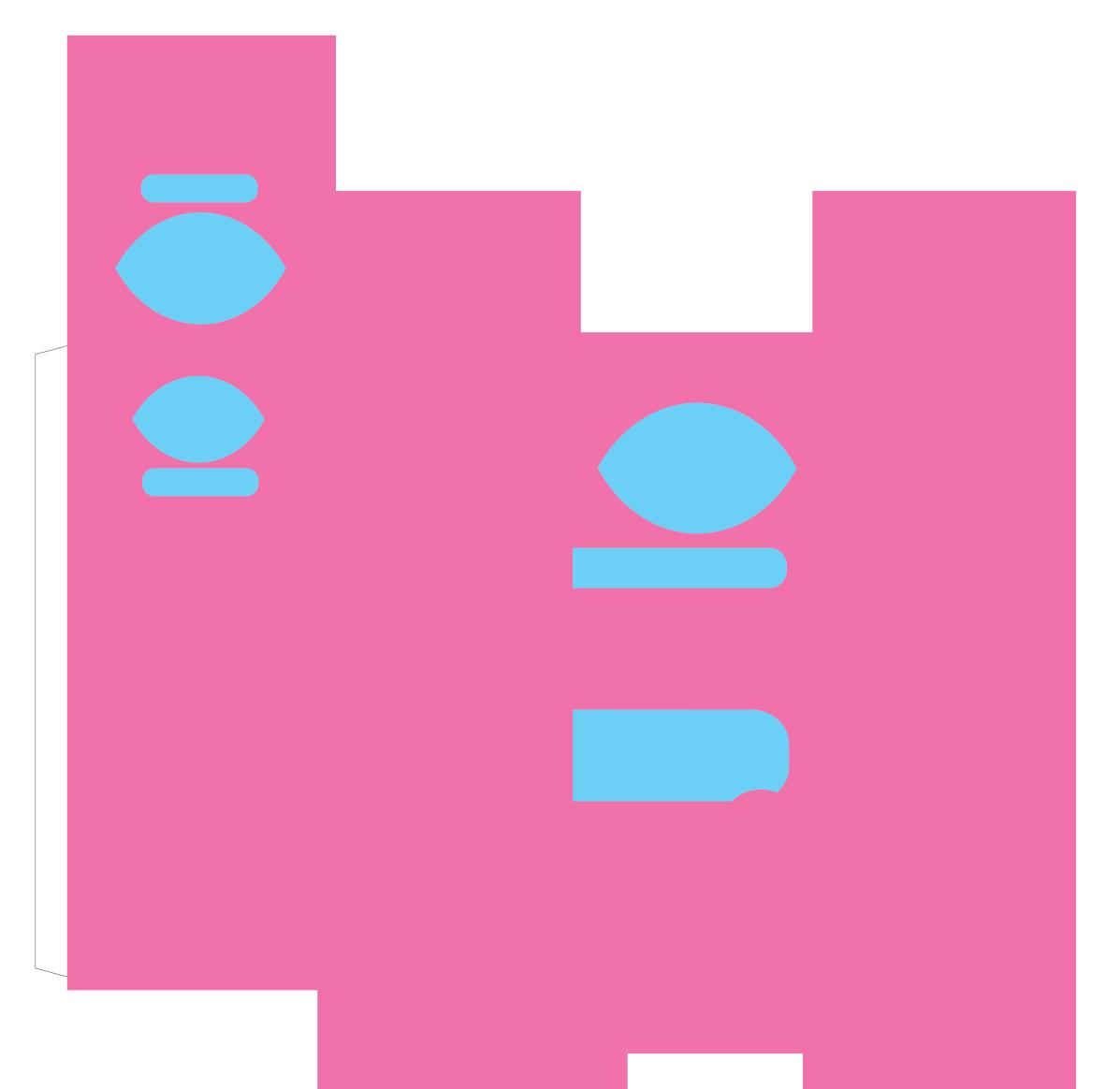



Approval Signature Approval. Whist we take extreme care at all times to ensure accuracy to our client's brief, the final responsibility must be taken by our client. JF YOU SIGN THIS PROOF YOU ARE SIGNIFYING FULL APPROVAL OF DESIGN AND TEXT.	Customer	East Midlands Pharma		Colours Used
	Item Code	PEMP312		Keyline (Non-Printing)
	Description	Vizulize Advanced Multi Purpose Contact Lens Solution 360	n a j	R蚯蚓 Touch Varnish
	Keyline Reference	EMP8		UV Varnish
	Varnish Reference	Drip off Primer Varnish		Cirrus_Info_Box
	Dimensions	70 x 70 x 180		
	Language	English		
	Market	UK		
	Proof By	CJE		
	Proof No.	2		
	Date	12/11/2020		