



40 Biodegradable Eyelid Wipes

Individually wrapped for daily eyelid hygiene



Suitable for
**Sensitive
Eyes**

40 Biodegradable Eyelid Wipes

Individually wrapped for daily eyelid hygiene

Cools, soothes, cleans & refreshes eyes

Preservative free + Detergent free

Dr Hilary Jones



The Eyelid Wipe is excellent for gently cleaning away debris from the eyelid and eyelashes.

Good quality tears operate best when a thin layer of oil lies over the tear surface. This seals the tears preventing excessive evaporation.

This oil comes from the glands in the eyelids. Daily eyelid hygiene is a key step in preventing various eye complaints and irritations.

- Dr Hilary Jones

40 individual Eyelid Wipes developed to clean your eyelids at home or on-the-go. An essential step in daily eyelid hygiene.

Directions for use

Wash hands. Remove wipe from sachet. Close the eye and wipe from nasal corner of the eye outwards across the lashes.

Environmentally friendly

The Eye Doctor® Biodegradable Eyelid Wipes are plastic free. Made from Viscose fibres, our wipes are fully biodegradable in natural environments: in the soil, compost and both fresh or marine water. They will fully convert back to nature and do not require recycling.

Ingredients Each individual wipe contains 99.1% of aqua (water) and 0.9% sodium chloride solution.

Warnings / Precautions

- Do not use if the sachet is opened or damaged
- Use a fresh wipe for each eye
- Dispose of wipe after use in the compost bin, do not flush down the toilet
- For external use only
- Remove contact lenses before use
- Store in a cool dry place
- If you experience any reaction to the use of these wipes, discontinue use immediately

Distributed By

The Body Doctor® Ltd, Unit 1,
Bridge Industrial Estate, Off Park
Mill Way, Clayton West,
Huddersfield,
HD8 9XJ, UK
+44 (0) 1484 868 816
www.the-body-doctor.com

EU RP: OMC NI Limited,
Unit 763, Moat House Business
Centre, 54 Bloomfield Avenue,
Belfast, NI, BT5 5AD
E: info@omcmedical.co.uk

For single use only - do not reuse

STERILE



MADE IN THE UK



REF 40D



5 034445 001242

Art-54-01-May23

3 Steps are required for eyelid hygiene

1

Heat eyelids externally

Using The Eye Doctor® Hot Eye Compress

2

Massage the eyelids

To cleanse the melted oil from the glands

3

Use the eyelid wipe

To clean the base of the eyelids and lashes

Suitable for use with

- + **Dry eyes**
- + **Sore eyes**
- + **Gritty eyes**

Symptoms can be helped by daily eyelid hygiene

Naturelle Consumer Products Ltd, Unit 5 Bankmore Way, East
Ind. Est. Omagh, Co Tyrone, BT79 0NZ, Northern Ireland, UK